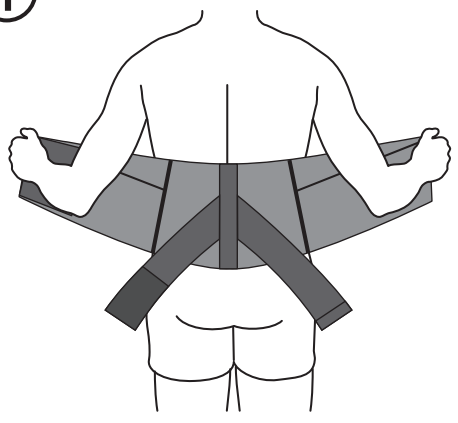
1

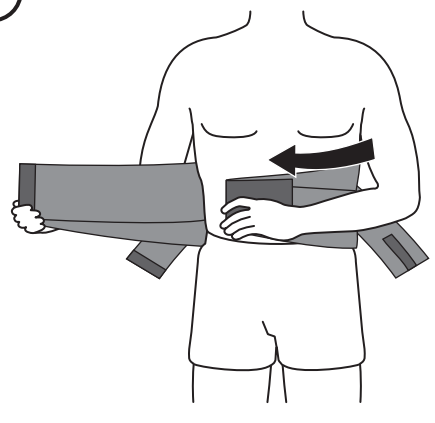
Center support on your back. Ensure large loop patch is on left side.

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2


Pull left side firmly across the middle section of your torso.

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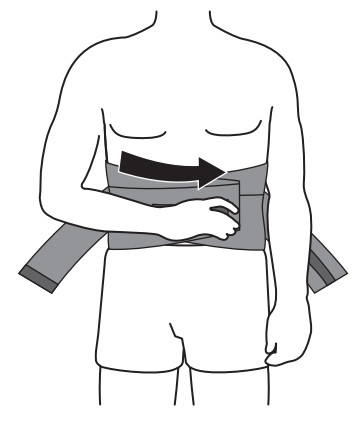3


Pull right side firmly across, and attach to the hook and loop closures.

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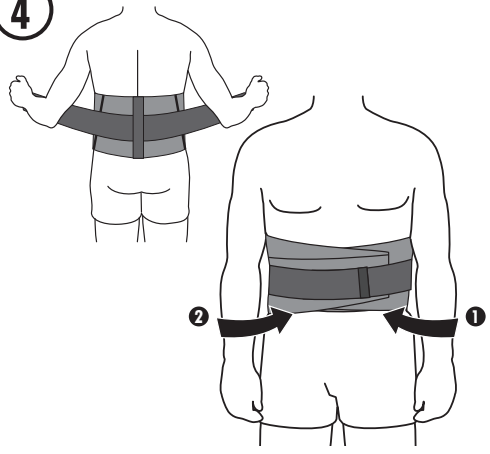4


Pull side pulls firmly around your torso. Attach left side pull first, followed by right side pull.

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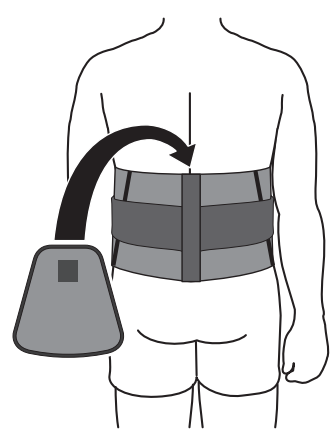5

If applicable, place the optional rigid insert into the open slot for added support

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## Back Supports Instructions for Use

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## WARNINGS

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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Breg, Inc.