WARNING: Carefully read application instructions and warnings prior to use.

WARNING: Do not remove this device unless instructed by your physician or medical professional.

WARNING: Do not over-inflate this product. Reduce air compression with any visual skin changes or reported discomfort. Do not use this device on patients incapable of communicating physical discomfort.

CAUTION: Use caution when walking on wet or slippery surfaces and/or uneven terrain to avoid injury.

WARNING: Do not operate a motor vehicle or human powered vehicle while wearing this device.

WARNING: Federal Law Restricts this Device to Sale by or on the Order of a Licensed Health Care Practitioner.

WARNING: For Single Patient Use Only.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device will not prevent or reduce all injuries.

CAUTION: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CAUTION: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65°C.) for any extended period of time.

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ENGLISH

This instructionsheet is intended for use after applying the boot to the patient.

INFLATING THE AIR BLADDER

After applying the boot to the patient using the application instructions provided, locate the quick pump bulb on the front of liner of the boot. Press the bulb to pump air into the bladder; after two pumps check the patient's ankle to see how the air bladder is fitting. The process of inflating the bladder may be done by the patient so that the patient controls the pressures in the bladder.

WARNING: Do not over inflate the Air Bladder!! Over-inflating the bladder may cause the bladder to burst and may stop blood circulation to the patient's foot. Over-inflating the Air Bladder will void its Warranty.

DEFLATING THE AIR BLADDER

To deflate the Air Bladder press the release button located above the air pump bulb, until the Air Bladder is flat and all the air has been removed.

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© 2015 Breg, Inc. All Rights Reserved