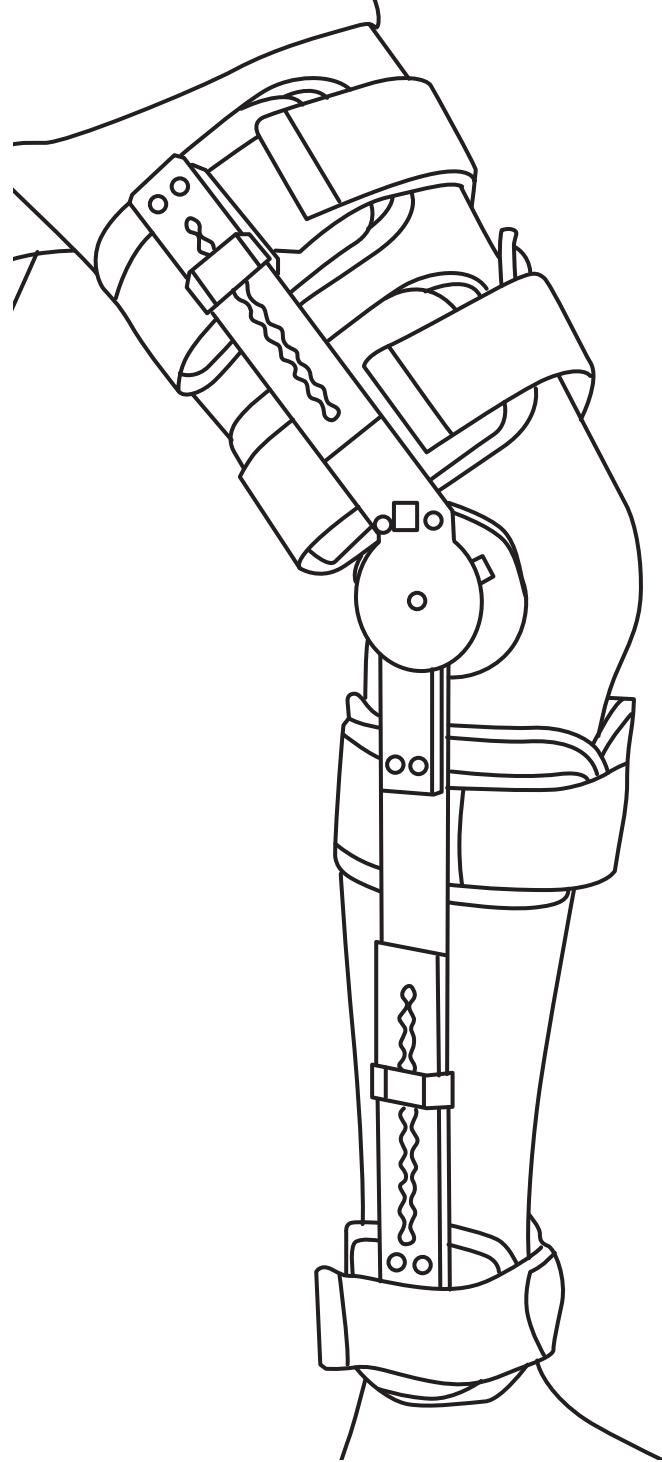
# Extender Plus Brace

INSTRUCTIONS FOR USE AW-CP020188 Rev C 1023

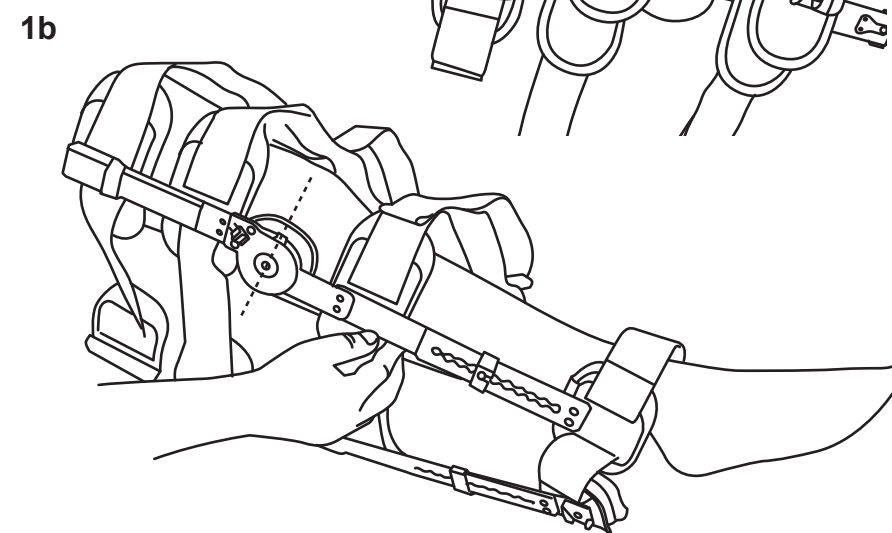
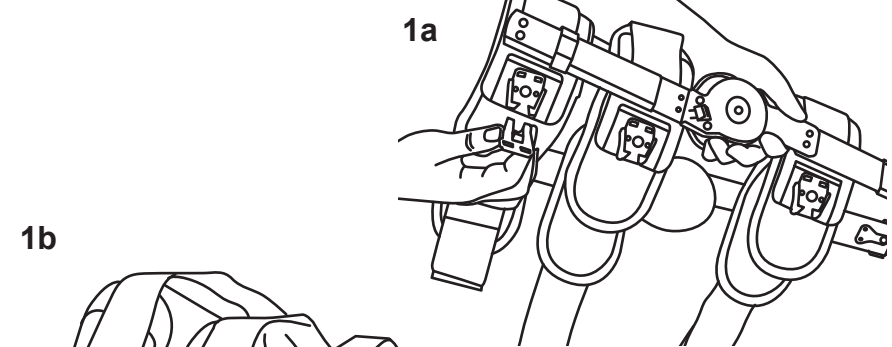
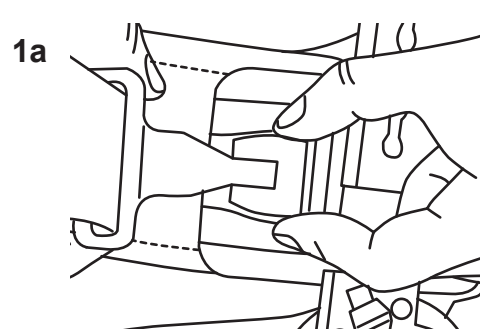
## FAST ADJUSTMENT - FAST HINGE - FAST STRAPS - FAST APPLICATION

© 2023 Breg, Inc. All Rights Reserved  
2382 Faraday Avenue, Suite 300  
Carlsbad, CA 92008 U.S.A.  
Toll Free Tel: 1-800-321-0607  
Local: +1-760-795-5440  
Fax: +1-760-795-5295  
www.Breg.com

E/U authorized representative  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

1. Release the over-center buckles.
  - a. Pinch the two buttons on the buckle releasing the buckle.
  - b. Place brace beside the leg aligning the hinge with the joint line.

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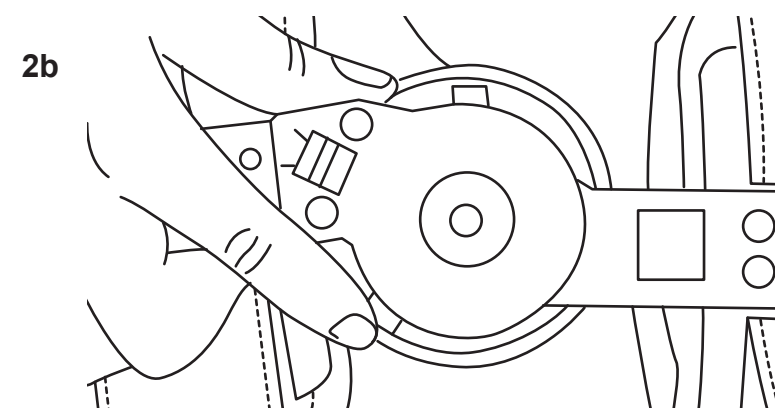
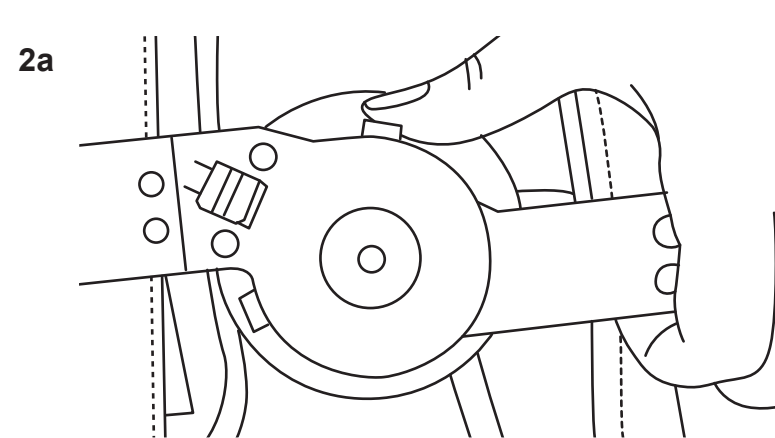
2. Set range of motion.
  - a. Depress extension ROM button and slide to desired setting.
  - b. Depress flexion ROM button and slide to desired setting.
 Settings automatically lock in place. Note: Refer to the Standard Hinge section of these application instructions for additional information on hinge adjustments.

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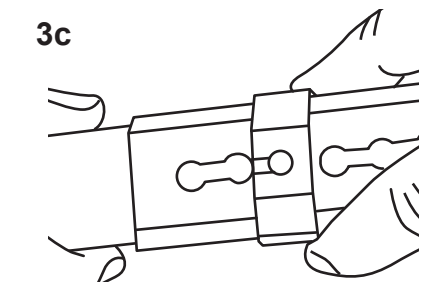
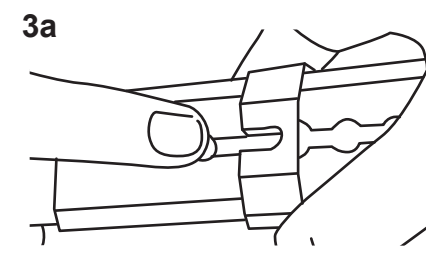
3. Set tibial and thigh length.
  - a. Slide the locking clip to free the length adjustment button.
  - b. Depress length adjustment button and slide tibial section until ankle plates are just above the ankle bone.
  - c. Slide the locking clip in place.
  - d. Repeat length adjustment for the thigh.
 Note: If using a shoe insert, apply it now by following instructions in the Extender Plus Shoe Insert section of these application instructions.

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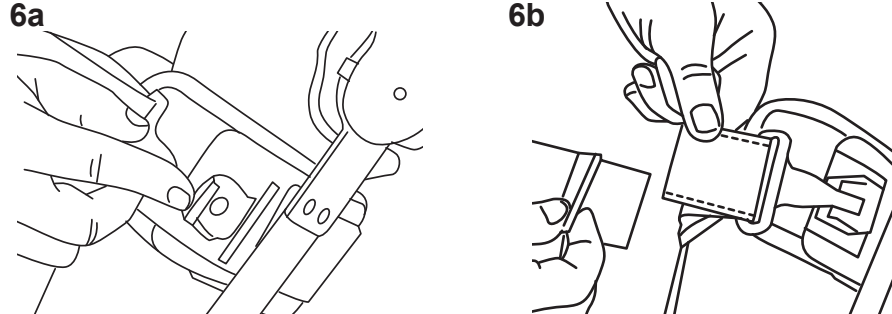
- 6.** Front strap adjustment.  
**a.** Attach the over-center buckle.  
**b.** Detach the hook and loop attachment from the strap end and tighten to desired tension.  
**c.** Trim off excess strap and attach the hook and pile attachment.

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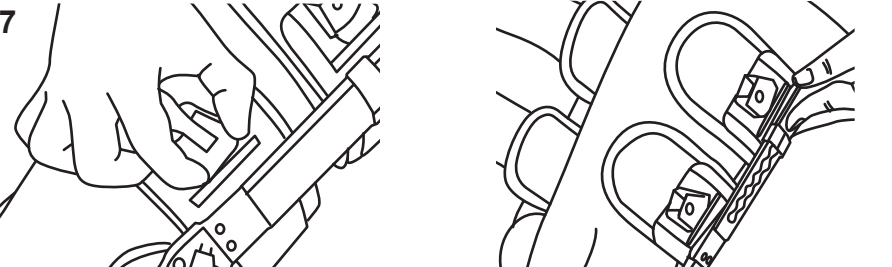
- 7.** Instruct patient. Show patient how to get out of the brace by pinching the tabs on the over-center buckles, and sliding leg out of the brace.

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**Standard Hinge Instructions**

- A.** Move uprights to the desired position between the prescribed range of motion (ROM).
- B.** Press the gray extension button and slide it to the desired setting. The button will spring out to set the extension range of motion.
- C.** Press the gray flexion button and slide it to the desired setting. The button will spring out to set the flexion range of motion.
- D.** To lock the brace at -10, 0, 10, 20, 30, or 40 degrees, move the uprights aligning the indicator mark with the prescribed setting in orange. Then slide the orange lock tab toward the hinge. This will lock the brace in place.

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**Shoe Insert Instructions**

- 1. Insert the key.** Complete steps 1 and 2 of the Extender Plus Brace application instructions. Insert key (provided) behind spring by depressing the button and sliding the key between the spring and upright.
- 2. Remove the tibial extension.** Turn the key 90°. Slide the tibial extension out.
- 3. Attach the shoe insert to the brace.** Slide the shoe insert into the brace and stop at desired length. Turn the key 90° and remove. Repeat for the other side.
- 4. Apply sock.** Place the sock (included with Extender Plus Shoe Insert) on the foot. Proceed to step 3 of the Extender Plus Brace application instructions.

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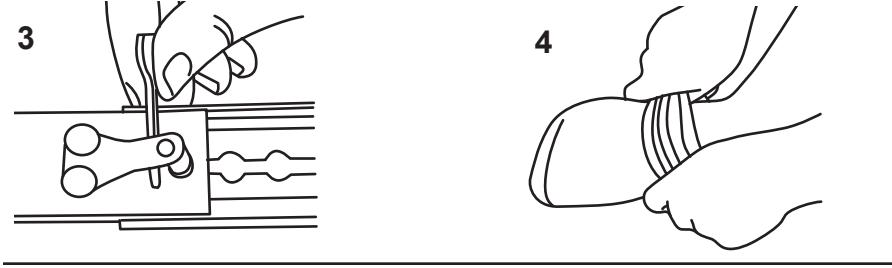
This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

**Indications:** Indicated for immediate application to knees for locked or limited motion control of knee during rehabilitation after operative procedures or injury to knee ligaments, cartilage, or stable or internally fixed fractures of the tibial plateau, condyles, or proximal tibia and distal femur.

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**Contraindications:** Contraindicated for unstable fractures or for fractures of the proximal femur or the distal tibia or fibula.

**WARNING:** Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, seek medical attention.**

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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