Orthopedic Support

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SUPPORTS ORTHOPEDIQUES

SOPORTES ORTOPÉDICOS

MD

CE

EC REP

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CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

ORTHOPEDIC SUPPORTS

- Hand wash and thoroughly rinse support. Air dry. To prolong product life and reduce skin problems, wash regularly.
- Hinge bars may be removed and cleaned. Normally, hinge lubrication is not required, however a dry lubricant, such as teflon spray may be used.
- After exposure to salt water, thoroughly rinse support in fresh water and air dry.
- Periodically inspect support for wear. Do not use if worn excessively.
- Do not wear support directly over open wounds.
- If you experience dermatitis or adverse skin reactions while using this product, consult your health care professional.

NOTICE TO USERS

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.