Plantar Fasciitis

Night Splint

FITTING INSTRUCTIONS

ENGLISH.....	4
SPANISH.....	5
ITALIAN.....	6
GERMAN.....	7
SWEDISH.....	8
DANISH.....	9
FINNISH.....	10
ROMANIAN.....	11
POLISH.....	12
DUTCH.....	13

Breg, Inc.


2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.

P: 800-321-0607





F: 800-329-2734

www.breg.com

©2022 Breg, Inc. All rights reserved.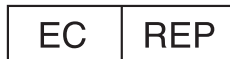

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

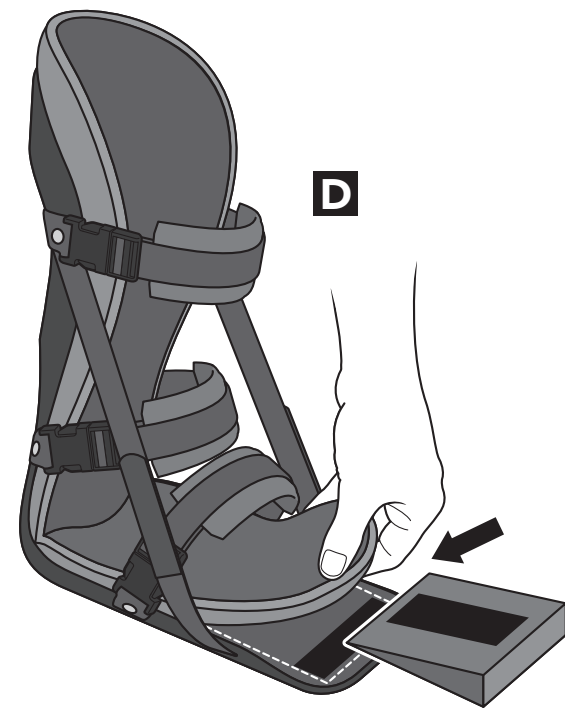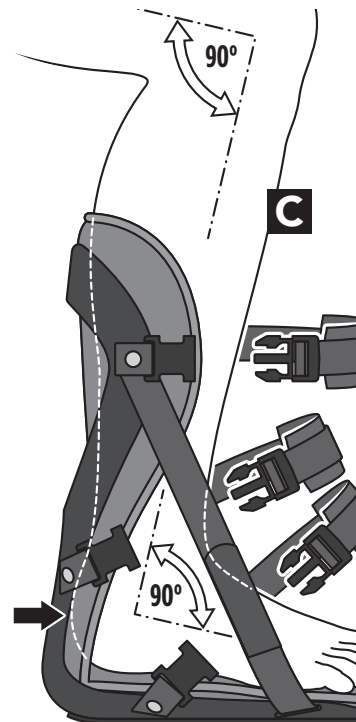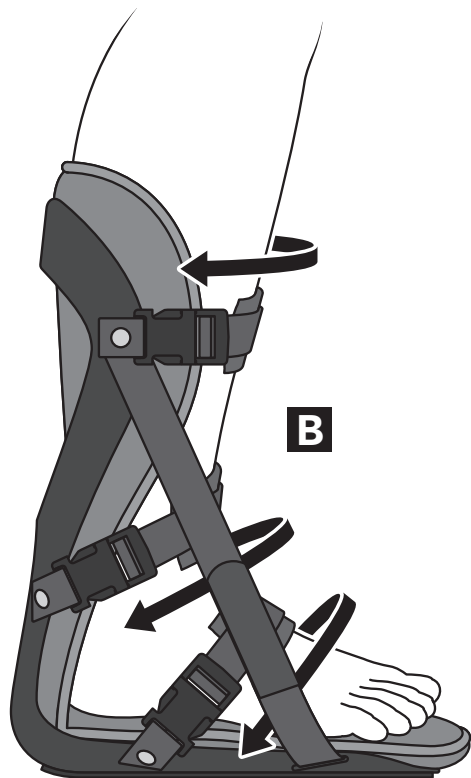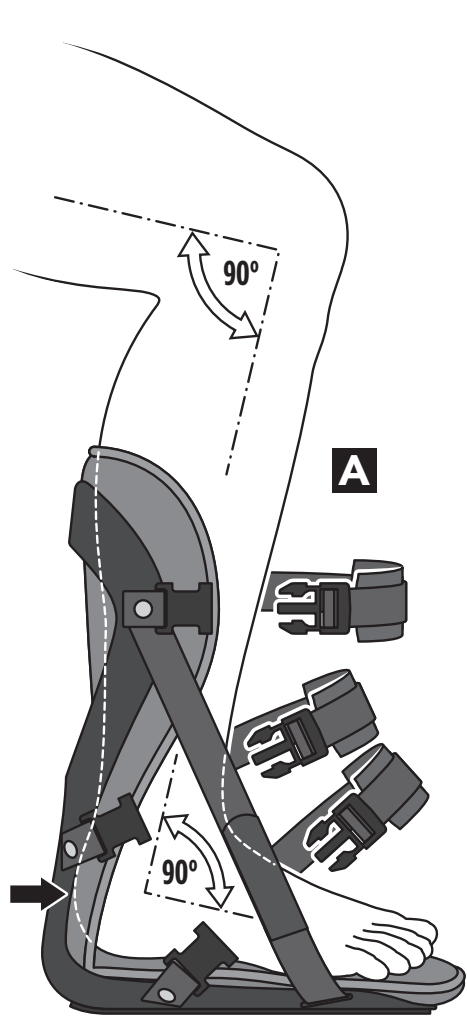⚠️ WARNINGS

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS.

WARNING: IF YOU EXPERIENCE OR HAVE AN INCREASE IN PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR MEDICAL PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: FOLLOW APPLICATION AND CARE PROCESS DESCRIBED IN INSTRUCTIONS.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CARE & WASHING INSTRUCTIONS:

REMOVE LINER FROM NIGHT SPLINT FOOT BASE. HAND WASH IN COLD WATER WITH A MILD DETERGENT, RINSE AND LAY FLAT TO AIR DRY.

- A.** Open all 3 buckles. Place the affected foot in the splint with knee flexed at 90°. The heel should be at the very rear of the splint and the toes should not extend past the end of the splint. The ankle should be at 90° when the splint is properly positioned.
- B.** Fasten the buckles. If necessary, adjust the buckles to ensure a secure and comfortable fit.
- C.** Release the hook and loop straps on each side of the foot bed. Pull up or lower the straps to adjust plantar fascia stretch.
- D.** To achieve additional stretch, pull the front portion of the liner away from the footplate and insert the 5° foam wedge between the liner and the footplate.

In the event of a serious accident in relation to the device, send a report to Breg and the competent authority of the Member State of residence of the user and / or patient.

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