

PREMIER THUMB SUPPORT

with STAYS

APPLICATION INSTRUCTIONS



1. Open thumb orthosis flat, place on affected thumb and wrap around thumb and secure thumb strap.
2. Wrap wrist strap snugly, securing to both hook areas along wrist area.
3. Re-adjust as necessary for comfortable fit.
4. Interchangeable flexible, rigid and malleable stays can be removed if desired.



Manufactured For:

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com

AW-1.95002 Rev D 1012

PREMIER THUMB SUPPORT

with STAYS

APPLICATION INSTRUCTIONS



1. Open thumb orthosis flat, place on affected thumb and wrap around thumb and secure thumb strap.
2. Wrap wrist strap snugly, securing to both hook areas along wrist area.
3. Re-adjust as necessary for comfortable fit.
4. Interchangeable flexible, rigid and malleable stays can be removed if desired.



Manufactured For:

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com

AW-1.95002 Rev D 1012