⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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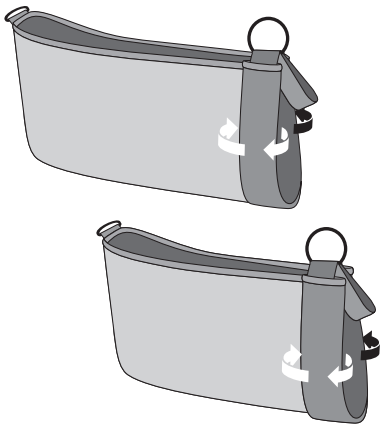
Universal Shoulder Slings and Immobilizers Instructions for Use

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1


To adjust the size of the universal sling, roll the cuff at the front of the sling to the correct arm length. Slide arm into the sling.

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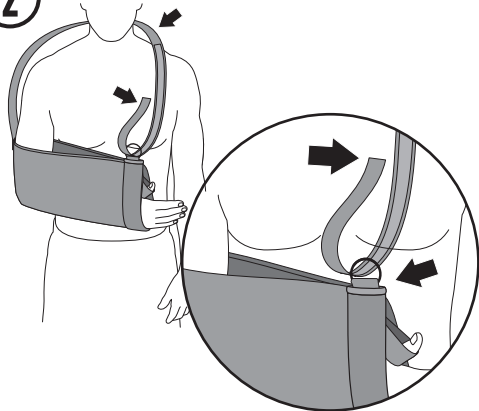2


Wrap shoulder strap around neck and shoulders. Lace strap end through O-ring and attach to shoulder strap.

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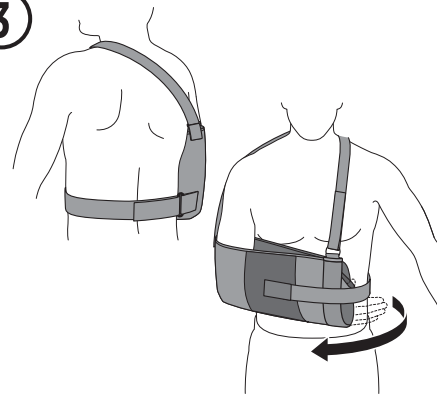3

Shoulder Immobilizer: After fitting the sling as indicated above, feed waist strap through plastic ring at the back. Wrap snugly around the waist and secure the strap to the Velcro® patch on the front of the sling.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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Breg, Inc.