## ⚠️ WARNINGS ⚠️

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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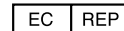
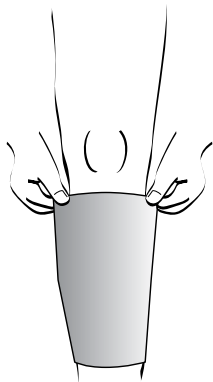
1

### 1. Apply Calf Sleeve

Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the calf so it is centered below the knee and above the ankle.

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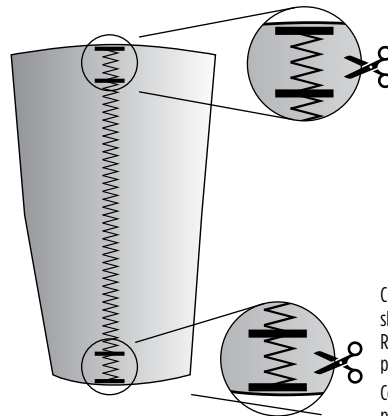
2

Cut between hem line stitch to shorten length (optional)

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### 2. Adjust length (optional)

To adjust length, cut between the additional hem line stitches at the top and/or bottom of the sleeve as shown.

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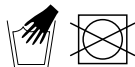
### CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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© 2017 Breg, Inc. All rights reserved.