⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Lateral Stabilizer Instructions for Use

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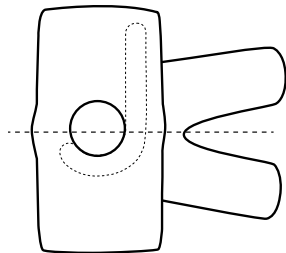
1

1. Apply Knee Sleeve

Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the knee so the opening is centered on the knee cap.

*Example illustrates left leg application.

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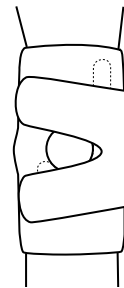
2

2. Secure Straps

First pull lower strap to attach, then pull upper strap to attach. Fit should be snug, but not too tight as to restrict circulation.

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Material: Neoprene or AirMesh®

CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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© 2023 Breg, Inc. All rights reserved.