English

1. Place the immobilizer under or behind the leg. Center the leg on the rear stay.
2. Adjust the removable panels laterally for correct sizing of thigh and calf circumference.
3. Once the immobilizer has been correctly adjusted, wrap the panels around the leg.
4. Starting closest to the knee, pull the straps through the D-rings and secure with the hook tabs provided. Then secure the remaining straps most distal and proximal on the immobilizer. Trim straps if necessary.
5. If applicable, secure the patella strap over the center of the knee and attach to the hook tabs both sides.

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Tri-Panel Knee Immobilizer Instructions for Use

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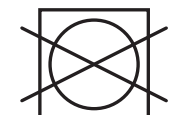
REV. B 07/23

WARNINGS

- WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- CAUTION:** Follow application and care process described in instructions.
- CAUTION:** For single patient use only.

CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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© 2023 Breg, Inc. All rights reserved.