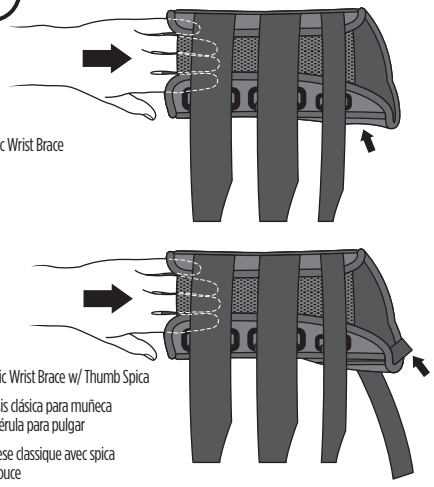
1

Classic Wrist Brace

Classic Wrist Brace w/ Thumb Spica

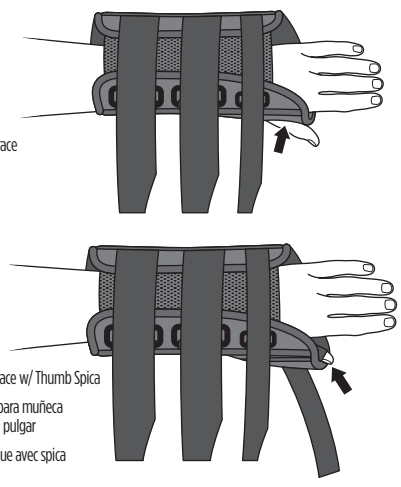
Órtesis clásica para muñeca con férula para pulgar
Orthèse classique avec spica du pouce

Classic Wrist Brace: Slide the brace onto affected hand. Be sure thumb and thumb hole on the side of the brace are aligned. **Classic Wrist Brace w/ Thumb Spica:** Slide the brace onto affected hand. Be sure thumb and elastic thumb loop are aligned.

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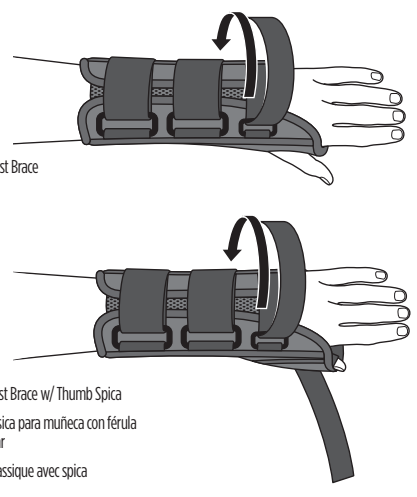
Classic Wrist Brace: Slide thumb into hole in the side of the brace. **Classic Wrist Brace w/ Thumb Spica:** Slide thumb into elastic thumb loop.

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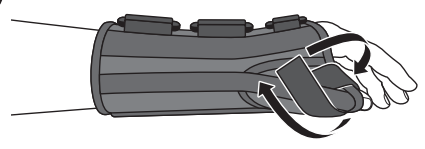
Thread the three closure straps through D-rings and secure to brace.

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4

Classic Wrist Brace w/ Thumb Spica: Wrap thumb strap around the injured thumb and secure to brace.

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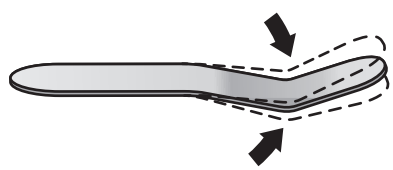
The curvature of the metal stay on the bottom of the brace should fit into the palm of the hand and extend down the forearm.

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If necessary, remove the malleable metal stay and bend it to the appropriate position.

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Trim hook and loop closure straps, leaving approximately 1" – 2" of overlap.

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Classic Wrist Brace Instructions for Use

Example illustrates left hand brace application.

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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© 2019 Breg, Inc. All rights reserved.