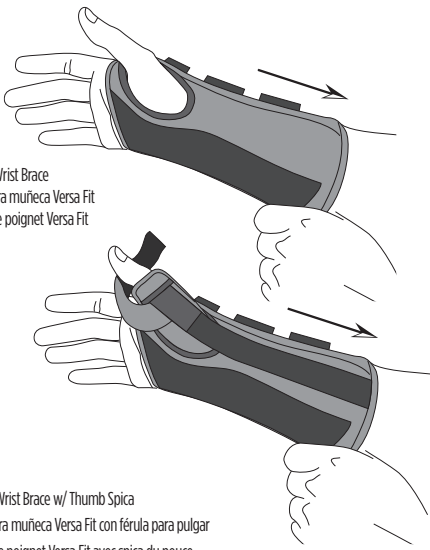
1

Versa Fit Wrist Brace
Órtesis para muñeca Versa Fit
Orthèse de poignet Versa Fit

Versa Fit Wrist Brace w/ Thumb Spica
Órtesis para muñeca Versa Fit con férula para pulgar
Orthèse de poignet Versa Fit avec spica du pouce

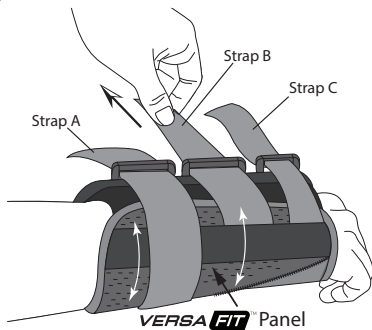
Versa Fit Wrist Brace: Slide affected hand into brace, gently guiding the thumb through the thumb hole. **Versa Fit Wrist Brace w/ Thumb Spica:** Slide affected hand into brace, gently guiding the thumb into the thumb support.

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4


Gently pull on the straps until the brace is taut; the Versa Fit panel will automatically adjust to the user's hand size. The proximal strap (A) locks the panel into functional size. **NOTE:** Do not overtighten. For best results, adjust the strap a second time to ensure a comfortable fit or continue to adjust as necessary.

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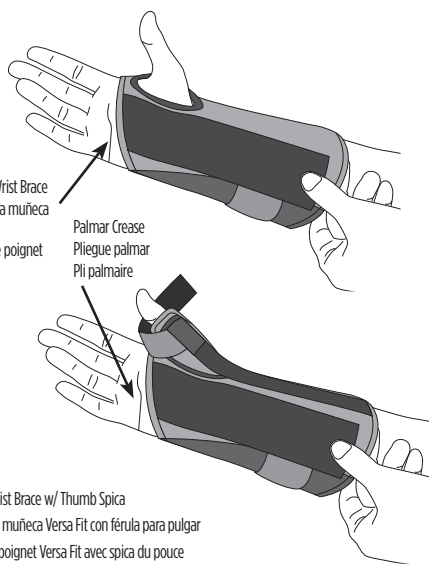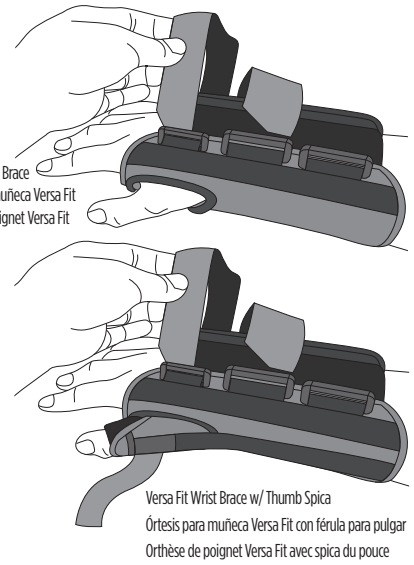
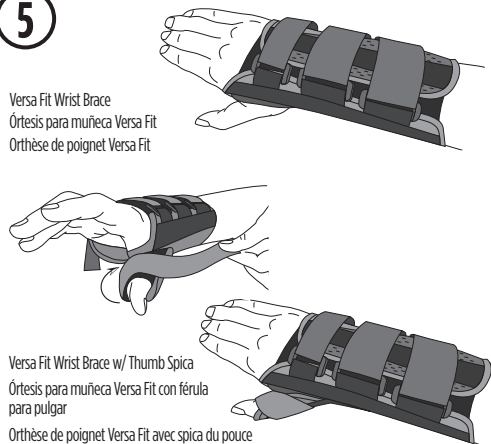
Thread loop straps through the D-rings.

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Versa Fit Wrist Brace: Final application. **Versa Fit Wrist Brace w/ Thumb Spica:** Apply loop thumb strap around thumb attaching to hook. Final application.

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Versa Fit® Wrist Brace Instructions for Use

Example illustrates right hand brace application.

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© 2019 Breg, Inc. All rights reserved.
Versa Fit is a registered trademark of Ortho Systems, Inc.

WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: If swelling increases or pain, numbness or skin irritation occurs, discontinue use and contact your medical provider immediately. This product is only a support device, not to prevent injuries or prevent recurrence of injuries.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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