


# CMC Thumb Guard

## Instructions for Use

Rx Only

### Application Procedure:

1. Measure width of patient's palm along the MP joints. Refer to chart below to select proper size. Slide hand into brace. Make sure strap is attached at palm side.
2. Pull straps around back of hand and through thumb/index finger space.
3. Pull strap around and below thumb joint.
4. Connect the strap to the brace by attaching the hook on the strap to the loop on the brace. Adjust for stability and comfort.

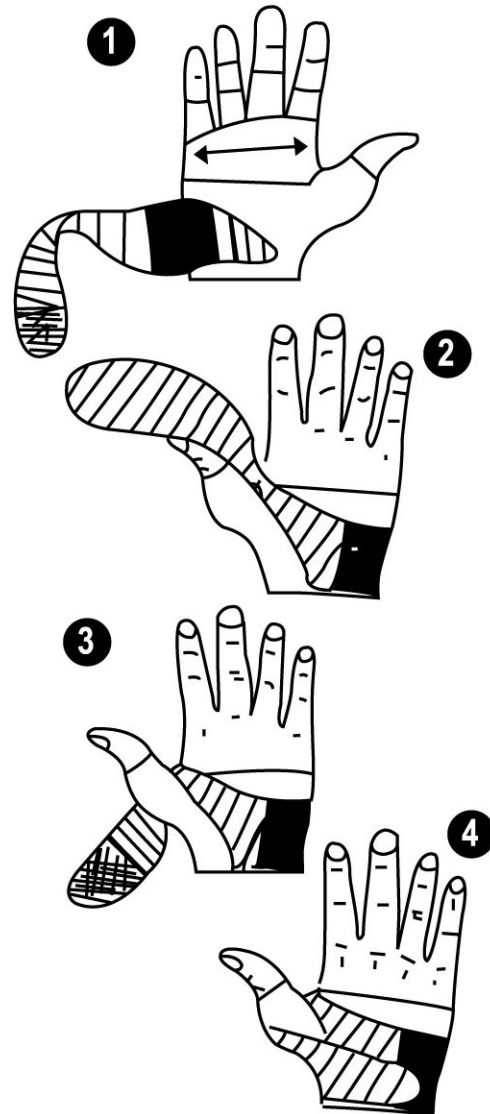

### Size Chart:

Size	Measurements
S	2 1/2" - 3"
M	3" - 3 1/2"
L	3 1/2" - 4"

Available for  
Left or Right wrists

**WARNING:** This device is intended for use only under the direction of a medical professional. If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, consult your medical professional immediately.



Breg, Inc.  
2885 Loker Ave. East  
Carlsbad, CA 92010 U.S.A.  
P: 800-321-0607  
F: 800-329-2734  
www.breg.com