

Part number: AW-1.03438  
Description: PC WAVE ICE BAG  
Revision: B 08/21

4.125 in

-----  
*Water Fill Line*  
-----


**Ice Bag Preparation**

- ❶ Fill 4 bags with water to Water Fill Line (approximately 2 cups).
- ❷ Freeze bags.

**Unit Setup**


- ❶ Add water to fill line in cooler.
- ❷ Place 4 frozen bags in corners of cooler.

**FIRST ADD WATER  
SECOND ADD BREG ICE BAGS**

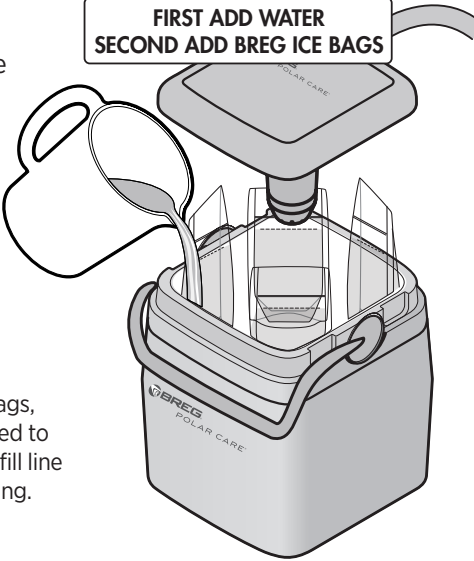

**Tip:**  
While using the Breg Ice Bags, cubed ice may also be added to the container up to the ice fill line to optimize the colder setting.

6.5 in



EC REP CE

E/U authorized representative  
MDSS GmbH, Schiffgraben 41  
D-30175 Hannover, Germany

©2021 Breg, Inc. All rights reserved.  
AW-1.03438 Rev B 08/21

PINK DOES NOT PRINT